



Cryo Cap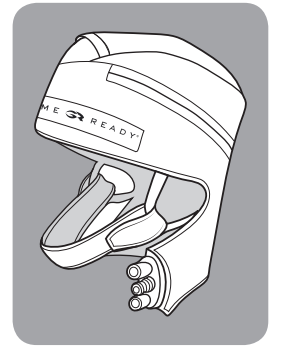


Application

1. Unfasten one side of the chin strap.
2. Place the cap on the top of the head and adjust the hook and loop closure at the front of the cap for a comfortable fit.
3. With the chin strap attached on one side, pull it down and around the front of the chin. Insert the other side of the chin strap into the ear loop and fasten the hook and loop closure (3a). The middle, widest point of the chin strap should be in-line with the center of the chin (3b). The chin strap should not be worn on the neck (3c).

English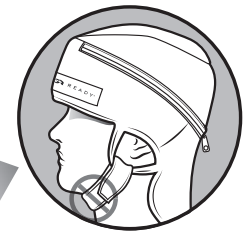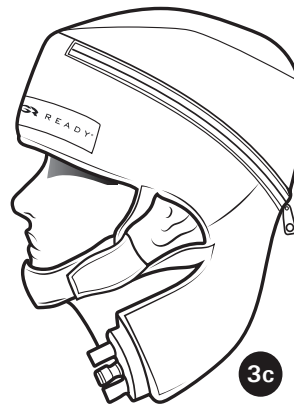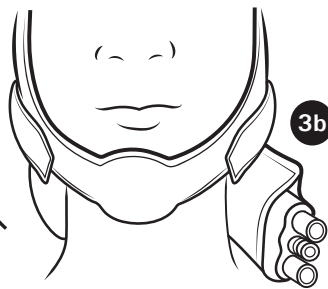


This Wrap comes with insulation layers for the ears and forehead. To use the wrap's insulation, tuck the foam layers under the Heat Exchanger in the desired region(s). To use the wrap without insulation, and provide more cooling for the ears and/or forehead, lay the foam on top of the Heat Exchanger in the desired region(s) (A).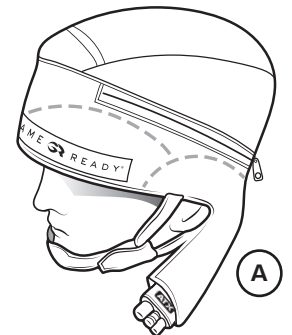


Assembly

1. Lay the Sleeve with the zipper side up on a flat surface and with the GR badge facing toward you.
2. Place the Heat Exchanger on a flat surface, separate from the outer Sleeve, with the blue side down.


- Take the two long, upper pieces of the Heat Exchanger and overlap them (3a). While overlapped, insert the Heat Exchanger into the Sleeve by sliding the two overlapped pieces into the center pocket of the Sleeve (3b). The blue side of the Heat Exchanger should touch the blue side of the sleeve.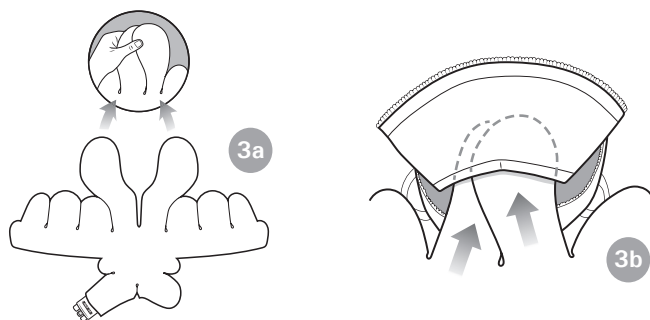

- Feed the ATX Connector through the connector opening on the side of the Sleeve. Tuck the neck flap into the pocket across from the connector.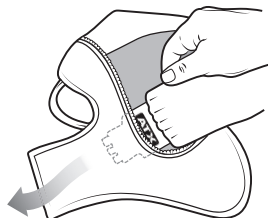

- Overlap the 3 wing-like pieces on one side of the Heat Exchanger (5a). Insert the wing-like pieces into the corresponding side wing space of the Sleeve. The wings will remain overlapped in the provided space (5b).

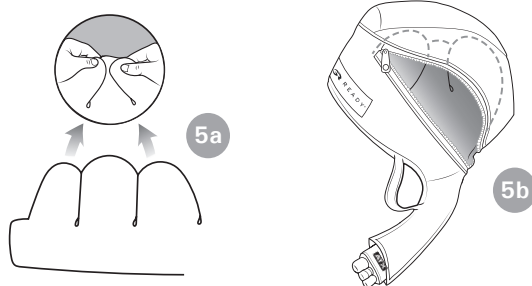- Once the first wings are in place, zip together that side of the Sleeve to keep the Heat Exchanger in place. Repeat the previous step for the opposite side of the Wrap.
- Zip the sleeve fully, making sure the Heat Exchanger is lying smoothly in the sleeve with no kinks or folds. Attach the hook and loop forehead closure.
- Feed one end of the chin strap through the ear loop and fasten the tab on the outside of the chin strap.

Important

This Wrap is intended for use with low-pressure setting. Pressure can be turned off as desired. Read this complete user's manual and Game Ready System user manual including indications, contraindications, cautions and warnings before using this product.

Contraindications

Compression Therapy using the Game Ready System or any compression therapy device should not be used in patients:

- Who are in the acute stages of inflammatory phlebitis in the affected region.
- Who have any current clinical signs suggestive of deep vein thrombosis in the affected region.
- Who have significant arteriosclerosis or other vascular ischemic disease in the affected region.
- Who have any significant risk factors or current clinical signs of embolism (e.g. pulmonary embolus, cerebral infarction, atrial fibrillation, endocarditis, myocardial infarction, or atheromatous embolic plaque).
- Who have a condition in which increased venous or lymphatic return is not desired in the affected extremity (e.g., carcinoma).
- Who have decompensated hypertension in the affected region.

Cryotherapy using the Game Ready System or any cryotherapy device should not be used in patients:

- Who have significant vascular impairment in the affected region (e.g., from prior frostbite, diabetes, arteriosclerosis or ischemia).
- Who have known hematological dyscrasias which affect thrombosis (e.g., paroxysmal cold hemoglobinuria, cryoglobulinemia, sickle-cell disease, serum cold agglutinins).

Warnings

Follow the treatment recommendations of your health care practitioner for duration and frequency of use for this device.

Improper placement or prolonged use of the Game Ready System could result in tissue damage.

During the course of therapy, patients should monitor the skin surrounding the treated region and the digits of the extremities of the treated limb (if applicable) for any burning, itching, increased swelling, or pain. If any of these signs are present, or any changes in skin appearance occur (such as blisters, increased redness, discoloration, or other noticeable skin changes), patients are advised to discontinue use and consult a physician.

Game Ready Wraps are not sterile; do not place directly against open wounds, sores, rashes, infections, or stitches. The Wrap may be applied over clothing or dressing.

Game Ready Wraps are available in multiple configurations but are not intended for all possible physiologic uses. For example, the Ankle Wrap is not designed for use on the toes and the Back Wrap is not designed for use in the abdominal region. Blood pressure should be monitored at cooling onset and at the end of the treatment session in patients who have hypertension.

Not to be used continuously for greater than 20 minutes per treatment session. This Wrap is supplied with insulating layers that can be positioned as desired to prevent excessive cooling of the ears and/or forehead. If you feel excessive cooling or discomfort, use additional insulation where needed or adjust the control unit temperature.

Follow recommendation of physician for appropriate use of insulating layer.

Compression Therapy with the Game Ready System should be used only under the supervision of a licensed healthcare practitioner in patients:

- Who have an open wound in the affected region (the wound must be dressed prior to use of Game Ready).
 - Who have an acute, unstable (untreated) fracture in the affected region.
 - Who are children under 18 years old or patients who have cognitive disabilities or communication barriers, whether temporary (due to medication) or permanent.
 - Who have a cardiac insufficiency or congestive heart failure (with associated edema in the extremities or lungs).
 - Who have a localized unstable skin condition (e.g., dermatitis, vein ligation, gangrene, or recent skin graft).
 - Who have erysipelas or other active infection in the affected region.
- Cryotherapy with the Game Ready System should be used only under the supervision of a licensed healthcare practitioner in patients:
- Who have Raynaud's disease or cold hypersensitivity (cold urticaria).
 - Who have hypertension or extreme low blood pressure.
 - Who have diabetes.
 - Who have compromised local circulation or neurologic impairment (including paralysis or localized compromise due to multiple surgical procedures) in the affected region.
 - Who have a localized unstable skin condition (e.g., dermatitis, vein ligation, gangrene, or recent skin graft).
 - Who have rheumatoid arthritis in the affected region.

Notes

Apply Wrap with a uniformly close fit, ensuring there are no kinks which may impede water flow. Ensure connector hose is placed to prevent the wrap from folding or kinking at the hose inlet location of the wrap.

Care and Cleaning

For daily care and to minimize formation of mildew, remove Heat Exchanger from Sleeve and wipe with dry towel to remove any condensation that may form. Turn Sleeve inside out and hang both the Sleeve and Heat Exchanger to release excess moisture.

For extended care, carefully remove the Heat Exchanger from the sleeve and turn the Sleeve inside out. Hand or machine-wash the sleeve in cold water and mild detergent, or antibacterial soap. Hang to dry. Heat Exchanger should be wiped with warm water and mild detergent-do not machine-wash or place in a dryer. Hang to dry.

For multi-patient use, Sterifab® may be used according to manufacturer's instructions on the Heat Exchanger and/or Sleeve to minimize microbe transfer.

Average Life Expectancy

The life expectancy of Sleeves and Heat Exchangers will vary widely depending on frequency of use. Please reference chart below to determine when to replace product.

Sleeve

Light Use (Personal)	12 months
Medium Use	6 months
Heavy Use (Clinic or training facility)	3 months

Heat Exchanger

Light Use (Personal)	24 months
Medium Use	18 months
Heavy Use (Clinic or training facility)	12 months

Warranty Information

Sleeve: In case of manufacturer defect, sleeve may be returned within 7 days of purchase.

Heat exchanger: 1 year from date of purchase. See warranty card included with Heat Exchanger.

Contact Us

In the U.S. call Game Ready Customer Service at 1.888.426.3732 (+1.510.868.2100). From outside of the U.S. please refer to www.gameready.com to find your local distributor's contact information

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©2016 CoolSystems, Inc. All Rights Reserved.