

**Welcome to Med4 Elite.** These simple steps will guide you through uncrating the Med4 Elite. Follow these steps to ensure a safe and speedy setup. We recommend having two people uncrate the Med4 Elite. To get started, you will need a power/battery operated phillips screwdriver.

NOTE: Screw location approximated in green, red, blue, orange, purple, and gray in each drawing.


**1 REMOVE TOP PANEL**  
 Remove the twelve (12) screws that secure the top panel in place. Carefully lift the top panel off of the crate.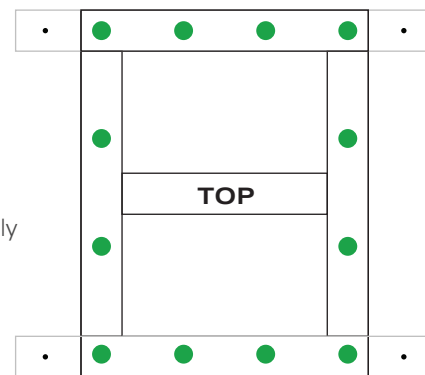


**2 REMOVE VERTICAL SCREWS FROM BOTH SIDE PANELS**  
 Remove the sixteen (16) screws (eight from each side) along each vertical edge of the side panels. **DO NOT** remove the interior or bottom screws from the side panel until instructed to do so.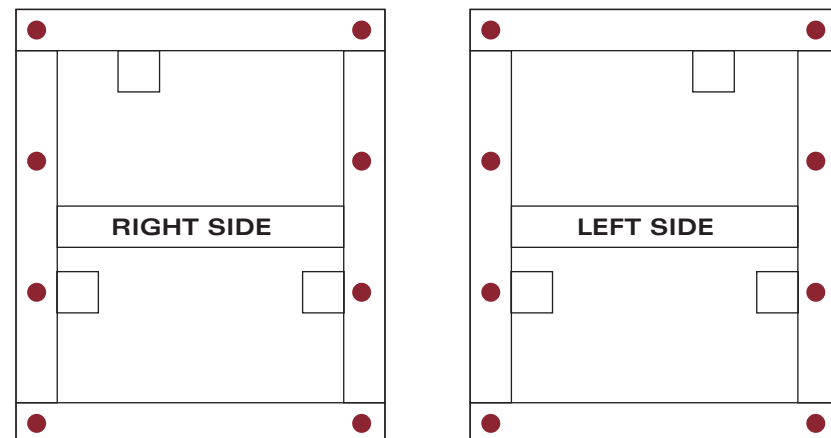

**3 REMOVE FRONT & BACK PANELS**  
 Remove the remaining eight (8) screws (four from each panel) that secure the front and back panels in place.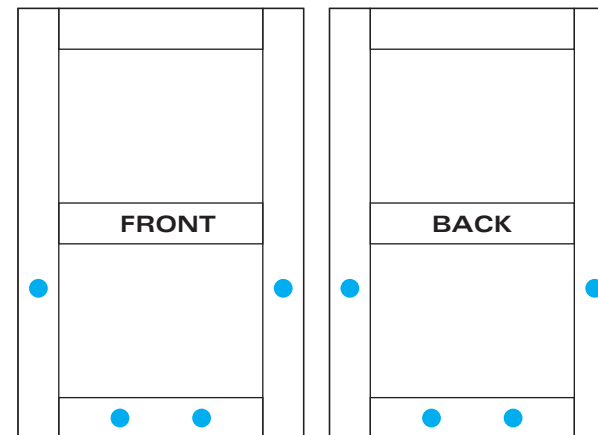

**4 REMOVE TOP BRACES**  
 Remove the eight (8) screws (four from each side) that secure the top braces in place. Carefully lift the braces away from the Control Unit.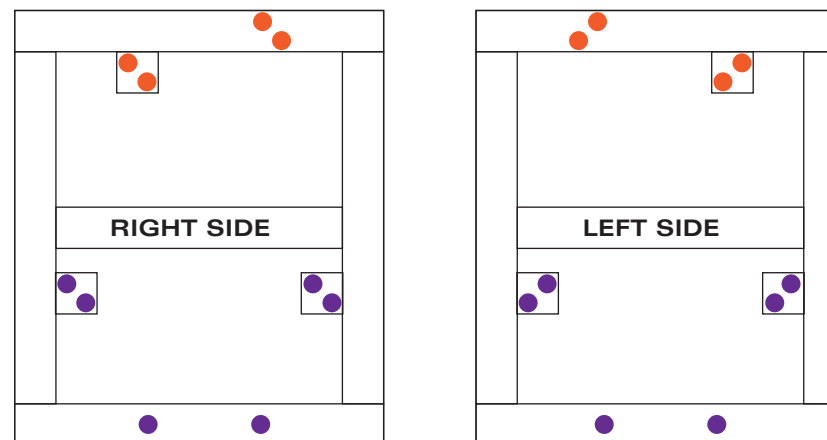


**5 REMOVE SIDE PANELS**  
 Remove the twelve (12) remaining screws (six from each side) that secure the side panels in place. Carefully lift the side panels away from the crate and set aside.

NOTE: Screw count is approximated. Always use caution when uncrating the Med4 Elite.

**6 REMOVE ACCESSORY KIT BOX**  
 The Accessory Kit box is located at the back of the Control Unit. Carefully detach it from the plastic protecting the Control Unit. Set it aside in a safe place. You will need the contents inside to begin setting up the Med4 Elite.

**7 REMOVE BOTTOM BRACES**  
 Remove the four (4) screws (two from each side) that join the bottom braces together. Carefully lift them away from the Control Unit.

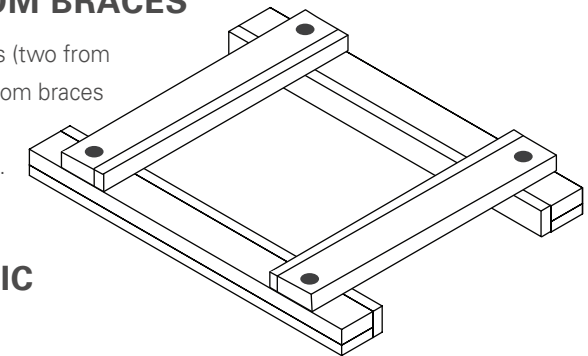**8 REMOVE PLASTIC COVER**  
 Remove the plastic protective layer and dispose of it.

**9 SET CONTROL UNIT OFF BASE OF CRATE**  
 Carefully lift the Control Unit off of the base of the crate and gently set down on a flat surface. Lock the casters. Two people are required to perform this step to prevent damage to the Control Unit. **Do not tip the Control Unit.**

**10 GETTING STARTED**  
 Use the Quick Start located inside the Accessory Kit box for instruction on the initial setup. We recommend saving the accessory box as storage for the drain hoses, USB flash drive, and Med4 Elite documentation. The Quick Reference cards are designed to hang from the handles of the Med4 Elite Control Unit.

Contact Game Ready Technical Support at:

**1.888.GameReady**

(1.888.426.3732) or 1.510.868.2100.  
 From outside the U.S., contact your local distributor.

# UNCRATING GUIDE

Contact Game Ready Technical Support at:

(1.888.426.3732) or 1.510.868.2100.

From outside the U.S., contact your local distributor.

COOLSYSTEMS®, INC.  
1800 Sutter Street, Suite 500  
Concord, CA 94520 USA  
1.888.GAMEREADY +1.510.868.2100

[www.gameready.com](http://www.gameready.com)  
[www.gameready.com/patents](http://www.gameready.com/patents)

© 2017 CoolSystems, Inc. All rights reserved.  
Med4 Elite Uncrating Guide PN 705153 Rev C

CE EC REP  
0086  
EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

EMERGO AUSTRALIA  
201 Sussex Street  
Darling Park, Tower II, Level 20  
Sydney NSW 2000  
Australia

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