

Welcome to Med4 Elite. These simple steps will guide you through uncrating the Med4 Elite. Follow these steps to ensure a safe and speedy setup. We recommend having two people uncrate the Med4 Elite. To get started, you will need a power/battery operated phillips screwdriver.

NOTE: Screw location approximated in green, red, blue, orange, purple, and gray in each drawing.


1 REMOVE TOP PANEL
Remove the twelve (12) screws that secure the top panel in place. Carefully lift the top panel off of the crate.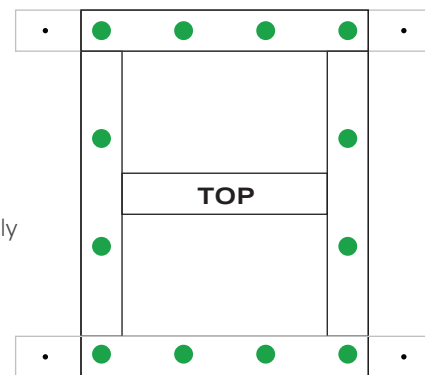


2 REMOVE VERTICAL SCREWS FROM BOTH SIDE PANELS
Remove the sixteen (16) screws (eight from each side) along each vertical edge of the side panels. **DO NOT** remove the interior or bottom screws from the side panel until instructed to do so.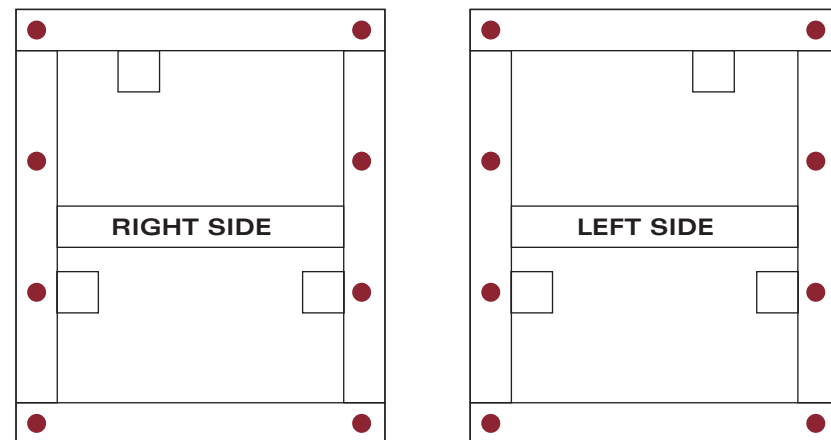

3 REMOVE FRONT & BACK PANELS
Remove the remaining eight (8) screws (four from each panel) that secure the front and back panels in place.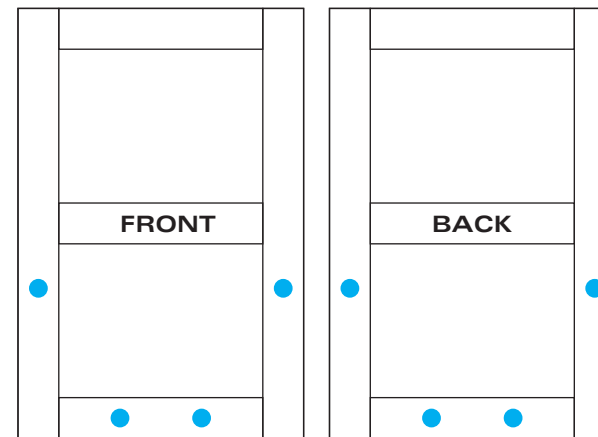

4 REMOVE TOP BRACES
Remove the eight (8) screws (four from each side) that secure the top braces in place. Carefully lift the braces away from the Control Unit.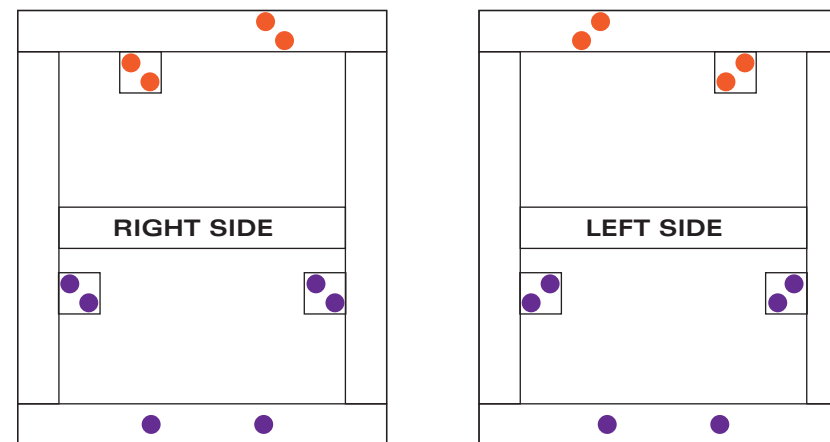


5 REMOVE SIDE PANELS
Remove the twelve (12) remaining screws (six from each side) that secure the side panels in place. Carefully lift the side panels away from the crate and set aside.

NOTE: Screw count is approximated. Always use caution when uncrating the Med4 Elite.

6 REMOVE ACCESSORY KIT BOX
The Accessory Kit box is located at the back of the Control Unit. Carefully detach it from the plastic protecting the Control Unit. Set it aside in a safe place. You will need the contents inside to begin setting up the Med4 Elite.

7 REMOVE BOTTOM BRACES
Remove the four (4) screws (two from each side) that join the bottom braces together. Carefully lift them away from the Control Unit.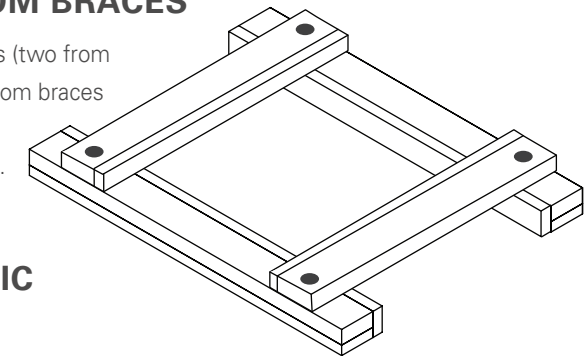

8 REMOVE PLASTIC COVER
Remove the plastic protective layer and dispose of it.


9 SET CONTROL UNIT OFF BASE OF CRATE
Carefully lift the Control Unit off of the base of the crate and gently set down on a flat surface. Lock the casters. Two people are required to perform this step to prevent damage to the Control Unit. **Do not tip the Control Unit.**

10 GETTING STARTED
Use the Quick Start located inside the Accessory Kit box for instruction on the initial setup. We recommend saving the accessory box as storage for the drain hoses, USB flash drive, and Med4 Elite documentation. The Quick Reference cards are designed to hang from the handles of the Med4 Elite Control Unit.

Contact Game Ready Technical Support at:

1.888.GameReady

(1.888.426.3732) or 1.510.868.2100.
From outside the U.S., contact your local distributor.

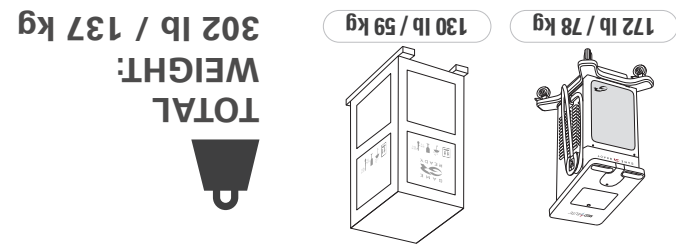UNCRATING GUIDE

Contact Game Ready Technical Support at:

1.888.GameReady

(1.888.426.3732) or 1.510.868.2100.

From outside the U.S., contact your local distributor.

COOLSYSTEMS®, INC.
DBA Game Ready®
1800 Sutter Street, Suite 500
Concord, CA 94520 USA
1.888.GAMEREADY +1.510.868.2100

www.gameready.com
www.gameready.com/patents

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

EMERGO AUSTRALIA
201 Sussex Street
Darling Park, Tower II, Level 20
Sydney NSW 2000
Australia

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© 2019 CoolSystems, Inc. 著作権所有