


14' Medical Connector Hose #570324

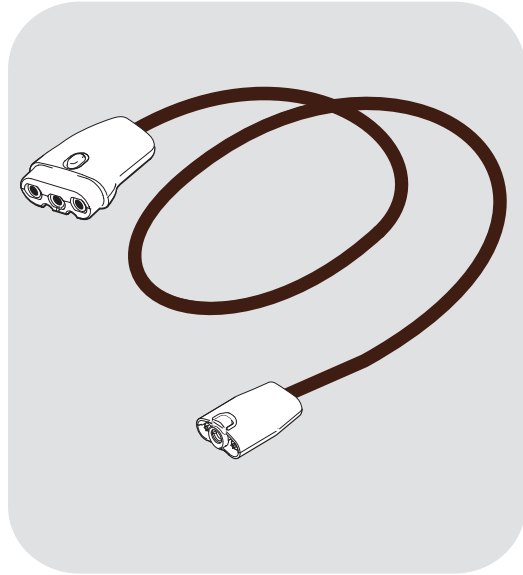

Game Ready 14' Medical Connector Hose


Warranty

The Game Ready Medical Connector Hose has a warranty of one year from the date of purchase.

Contact Game Ready Customer Service with questions or for further information at 1.888.Game Ready (1.888.426.3732).



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

 COOLSYSTEMS®.INC.
DBA Game Ready®
1800 Sutter Street, Suite 500
Concord, CA 94520 USA
1.888.GAMEREADY +1.510.868.2100

www.gameready.com
www.gameready.com/patents

704462 Rev C

14' Medical Connector Hose #570324

Game Ready 14' Medical Connector Hose

Cleaning Instructions

The surface of your Game Ready Medical Connector Hose may be wiped down as needed using soap and water.

To decontaminate the surface, certain commercially available agents may be used, always following the instructions provided with the agent. We recommend Quaternary Ammonia based cleaning agents, such as Cavicide®. When possible, we recommend the use of Caviwipes® or equivalent. A liquid form may also be applied using a soft cloth. In all cases, follow the cleaning agent manufacturer's instructions to determine the necessary exposure time for proper disinfection.

Warnings

- Do not use any harsh or abrasive materials on the Medical Hos
- Do not use any phenolic based cleaners on the hos
- Do not subject the hose to any sterilization protocol (i.e. autoclave or ethylene oxide sterilization)

Maintaining Hose Connection Points

Your hose connection points may require lubrication from time to time. Use only silicone based lubricants as **petroleum based products will destroy the components of the Connector Hose and void your warranty.** You can obtain silicone spray from Game Ready by contacting Customer Service at 1.888.Game Ready (1.888.426.3732).

704462 Rev C